ORDER INFORMATION
BESTELLINFORMATION

| Sizes / Größen | Content / Inhalt | Art. No. / Art. Nr. | |
|----------------|------------------|---------------------|-------------|
| | | Neuro-Patch® | Lyoplant® * |
| 12 x 14 cm | 1 piece / Stück | 106 4002 | - |
| 6 x 14 cm | 1 piece / Stück | 106 4010 | 106 6021 |
| 8 x 9 cm | 1 piece / Stück | 106 4020 | 106 6030 |
| 6 x 8 cm | 1 piece / Stück | 106 4029 | 106 6242 |
| 5 x 6 cm | 1 piece / Stück | 106 4040 | 106 6050 |
| 4 x 10 cm | 1 piece / Stück | 106 4037 | 106 6048 |
| 4 x 5 cm | 2 pieces / Stück | 106 4045 | 106 6064 |
| 2 x 10 cm | 2 pieces / Stück | 106 4053 | 106 6080 |
| 1.5 x 3 cm | 2 pieces / Stück | 106 4061 | 106 6102 |

* To improve smoothness and flexibility, Lyoplant® material is rehydrated prior to implantation.

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SUTURABLE DURA SUBSTITUTION
VERNÄHBARER DURAERSATZ

MODE OF APPLICATION

To achieve tension-free embedding, Neuro-Patch® and Lyoplant® should be cut as closely as possible to the defect size.

Neuro-Patch® and Lyoplant® should be fixed with non-absorbable suture material (polyester, polypropylene). Additionally, it may be secured with fibrin glue.

Atraumatic round-bodied needles allow suturing without significant damage to the implant.

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The main product trademark "Aesculap" and the product trademarks "Lyoplant" and "Neuro-Patch" are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

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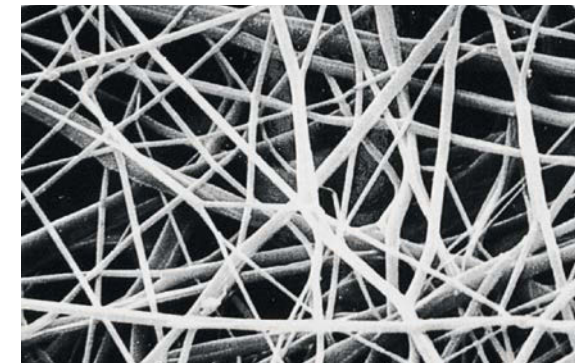
AESCULAP® Neuro-Patch®

SYNTHETIC, SUTURABLE DURA SUBSTITUTION
SYNTHETISCHER, VERNÄHBARER DURAERSATZ


ELASTIC. WELL-TOLERATED. NON-ABSORBABLE.

Neuro-Patch® is a type of fine fibred microporous fleece manufactured from a highly purified polyesterurethane.

An electronically controlled procedure has been developed for producing Neuro-Patch®, in which a dissolved polyesterurethane polymer is sprayed from specialised jets. This produces fibres which are captured at defined angles, producing a fleece-like structure.

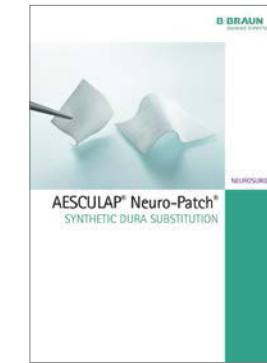The fine fibred microstructure supports the rapid infiltration of connective tissue cells.

C87802

Product sample card including a product sample.

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BIOLOGICAL, SUTURABLE DURA SUBSTITUTION
BIOLOGISCHER, VERNÄHBARER DURAERSATZ

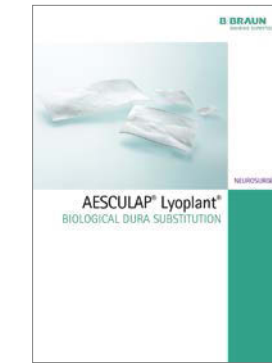
FUNCTIONAL. TEARPROOF. ABSORBABLE.

Lyoplant® is a pure collagen implant that is produced from bovine pericardium. This highly purified collagen is known for its low propensity to cause immunological reactions.

The special and comprehensive manufacturing process eliminates all non-collagenous substances such as fats, enzymes and other non-collagenous proteins.

A controlled lyophilisation (freeze drying) process supports that the loose fibre structure of Lyoplant® is preserved to offer appropriate conditions for integration after implantation.

After implantation, Lyoplant® slowly degrades enzymatically and is substituted by body's own connective tissue.

C87902

Product sample card including a product sample.

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PRODUCT CHARACTERISTICS¹

- High liquid tightness of the implant prevents CSF leakages
- Tissue tolerant
- High tensile strength of the implant prevents suture pull out
- Good suturability
- Adaptable to anatomical structures

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¹ Data on file, Aesculap AG.